

Micro-Shield™



by ZANSORS®

C
O
M
P
O
N
E
N
T
S




Mask



mask-locks for adjustable fit

Instructions for Micro-Shield™


1. Tear top of large silver package and pull out Mask and mask-locks pack

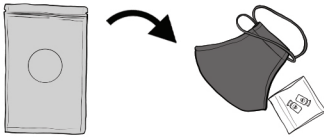

2. Remove mask-locks from their pack



3. Pinch Mask string and thread into mask-lock to desired fit

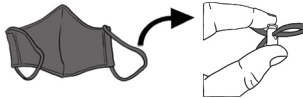

Important Information

Putting on the mask

• Make sure all sides are well sealed to your face and adjust the embedded, bendable nose wire for tighter fit.



Breathing experience

• When you breathe in and out the mask may move back and forth slightly. This is normal.

See www.ZShieldMask.com for more information on product use, FAQs, and other resources.

Follow and share
@ZShieldMask



WARNING Misuse may result in sickness or death. The decision to use this device is solely your own. Manufacturer has made no express or implied warranties related to this mask.



Micro-Shield™ by Zansors

- Thank you for purchasing Micro-Shield™ as your face mask
- Your mask is reusable. Clean it whenever needed. See instructions below
- Micro-Shield™ uses Cool-Fabric™ to promote air flow and keep you cool and comfortable
- Cool-Fabric™ provides silver ion technology, UV protection, and anti-sweat properties
- Embedded metal nose-clip and adjustable straps allow tighter seal

Wash and Care



Cool-Fabric™ Mask

- Use water only or water and soap to wash the mask by hand for 30 seconds.
- Gently squeeze excess water from the mask and air dry for at least 10 minutes. (Please be careful not to damage the internal wire nosepiece during removal of excess water.)
- Users can also use ethanol based sprays to disinfect the mask and let it air dry.
- Another option is to wash the mask in the washing machine



- **Micro-Shield has been authorized by FDA under an Emergency Use Authorization (EUA) for use by the general public and health care personnel (HCP) as source control to help prevent the spread of infection or illness during the COVID-19 pandemic.**
- This product has not been FDA cleared or approved
- This product is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under Section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.